

MEDRAD MRI PUMP INSTRUCTIONS

PA Catheters and Horizon Pump tubing are **NOT MRI Compatible**
You must bring medication drip and Primary tubing with you



Pump must be kept in charger when not in use.

Never take charger in magnet room

To remove from charger press here

Battery will last 4 hours fully charged.

Fully depleted battery takes 6 hr to recharge


Pump must be in the mounting bracket for infusion.

DO NOT lay the pump in the scanner with patient.

Pole and pump are MRI compatible right up to the magnet

- 1) **Attach Medication bag with PRIMARY Tubing to MEDRAD tubing.**

DO NOT USE HORIZON PUMP TUBING !
(Metal in the tubing is not MRI compatible)

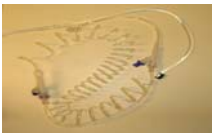

MEDRAD TUBING kept in MRI with pump

Tubing not recommended with Nitroglycerin or Blood



- 2) Open door: latch on right side

- 3) **Place black tubing fitting into the pump.**

Tubing must be in pump to prime. NO free flow

- 4) Close door and turn on pump



- 4) **PRIME TUBING**

Press **ORANGE PRIME** key

Press **Start OK**

Press **STOP NO** when prime is complete or pump will prime 40cc automatically

TO BEGIN INFUSION

Set **RATE** with # keypad

Press Green **START OK**

Set **VOLUME** of bag

Press **START OK**



Infusing indicator:
&
Green light
flashing

To titrate while infusing

Change the rate with the # keys

Press **START OK**

(Do not stop pump to change rate)

Information key (i) key

Press once = Volume infused

Press twice = Battery level

CLEAR VOLUME BEFORE TURNING PUMP OFF !

1) Press **STOP NO**

2) Press **i** key to display volume infused.

3) Press **STOP NO** to clear volume



Press release button

WHEN FINISHED

Press button to remove pump
from mounting bracket

Return to Charger

